

# PREvail® Wipes

## QUICKSTART GUIDE



NEW!  
Refill Wipes

### DAILY USE DISINFECTANT

VIRUCIDAL, BACTERICIDAL,  
FUNGICIDAL,  
TUBERCULOCIDAL

KILLS  
PATHOGENS IN



Prevail Wipes are ideal for daily cleaning and disinfection within animal care settings.

Excellent for fast and easy disinfection of high-touch surfaces and equipment.

### How to Use Prevail™ Wipes

#### STEP 1

##### Pre-clean visibly soiled areas


Remove visible soil if required. Pre-clean the surfaces by working from the least contaminated area to the most contaminated area.



#### STEP 2

##### Apply Prevail™ Wipes to the surface


Wipe the surface, making sure the surface is visibly wet.

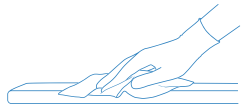

#### STEP 3

##### Allow surface to remain visibly wet for 1 minute

Allowing surfaces to remain wet for 1 minute kills viruses, bacteria and fungi per label instructions.

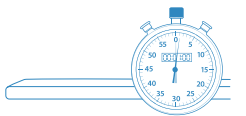

WIPE or AIR DRY

#### STEP 4

##### Wipe or allow to air dry

### Tips for Success

#### Refill Reminder

Refill your canisters with Prevail Refill Wipes. A convenient option that is cost-effective, compact, and made with 90% less plastic than regular canisters.

#### Close the Lid

Prevail™ Wipes have a shelf life of 3 years. Make the most of that time by remembering to close the lid of the canister after each time you pull a wipe.

#### Rinse Your Gloves

You may detect a rotten egg smell when wearing gloves. The odour is a harmless reaction between sulfur residue on the gloves and hydrogen peroxide in Prevail™.

You can:

- Continue to use the gloves
- Switch to a different kind of glove
- Rinse the surface of the gloves with water before using

#### Streak Free Finish

If desired, wipe the surface with a damp, clean cloth *after the contact time is met*. This will leave a streak free surface.

For more resources visit [PrevailDisinfectants.ca](http://PrevailDisinfectants.ca)



Prevail™ is a member of the Virox family of brands.